Use Instructions

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For nearest distributor or further assistance, contact: 4125 Westcor Court, Coralville, IA 52241, USA







Phone: (319)545-7100 • Fax: (319)545-7099

E-mail: info@protekmedical.com

Web: www.protekmedical.com

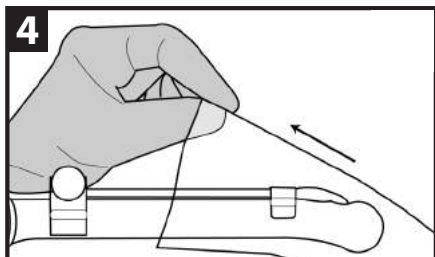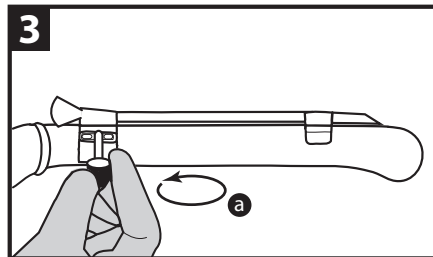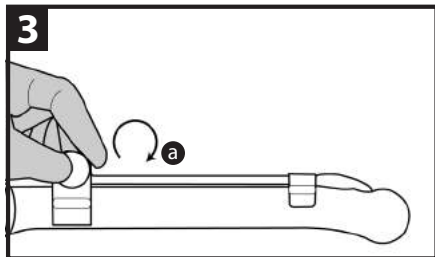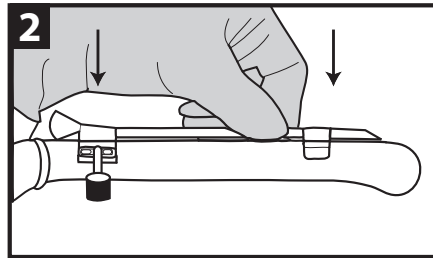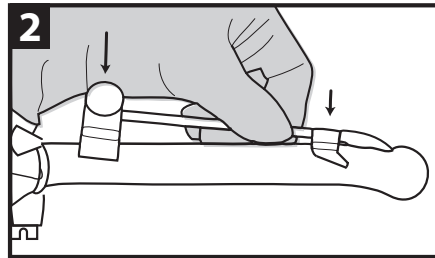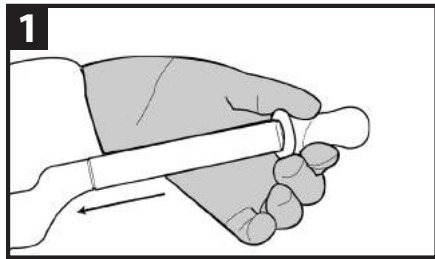CE 0086

USE INSTRUCTIONS

Prepare probe per system's operational manual. Reusable needle guide is provided non-sterile. Always verify needle follows guidelines prior to use.

 If needle guide does not fit properly, do not use. Clean/sterilize needle guide before use, following recommended instructions below. Use appropriate needle length to reach target area.

REUSABLE NEEDLE GUIDE

- 1** Place gel inside cover and/or on probe scanning surface. Place probe cover over probe. Secure cover with bands.
- 2** Place needle guide (styles vary) onto probe as shown.
- 3** Make sure needle guide is securely attached to probe (a).
- 4** Place second probe cover over probe and needle guide. Secure cover with bands.

 After use, dispose of single-use components as infectious waste.

CLEANING, DISINFECTING & STERILIZING REUSABLES

Cleaning

Reusable devices must be thoroughly cleaned before they are subjected to the sterilization process. Remember, you cannot achieve sterilization or high-level disinfection unless the assembly is cleaned first.

1. Rinse excess soil from device.
2. Prepare the enzymatic solution (like Enzol), following manufacturer instructions for proper dilution.
3. Fully immerse the device in the detergent and allow soaking for 10 minutes.
4. After the soak, brush device thoroughly using a soft bristle brush. Clean lumens and holes using an appropriate size brush. A syringe might be used to flush the hard-to-reach areas of the device.
5. Thoroughly rinse and dry the device. Always use R/O (reverse osmosis) or DI (de-ionized) water for rinsing the device, tap water may recontaminate the reprocessed device. Use a clean soft cloth to dry the device.
6. Thoroughly examine all surfaces to make certain that no visible bioburden remains. If any visible parts observed on the device, repeat cleaning and drying, until the ENTIRE device is clean.

Recommendations for High Level Disinfection/Sterilization of Plastic or Anodized Aluminum Reusables

After cleaning by using the above recommendations, soak for a minimum of 5 minutes in a neutral pH Enzol Enzymatic detergent such as made by *Johnson & Johnson*®.

Sterilize with Cidex Activated Dialdehyde solution such as made by *Johnson & Johnson*®, or any equivalent validated sterilization method adopted by the sterilization department for like devices.

Steam Sterilization of Stainless Steel Reusables

After cleaning by using the above recommendations, steam sterilize (sterilizer type - gravity) in an autoclave at minimum 132 degrees Celsius (270 degrees Fahrenheit) for a minimum of 15 minutes. Dry the device for a minimum of 25 minutes. When sterilizing multiple instruments in one autoclave cycle, ensure that the sterilizer's maximum load is not exceeded.

Warning

- Long narrow cannulations and blind holes require particular attention during cleaning.
- Do not reuse or reprocess devices labeled for single-use.
- After sterilization, appropriately package and store the device to ensure that sterility is not compromised prior to reuse.
- Always use a probe cover to minimize any contaminants.

Inspection and Function Testing

- Visually inspect for damage and wear. For hinged instruments, check for smooth movement of hinge without excessive "play." Locking mechanisms should be checked for action.

Limitations on Reprocessing

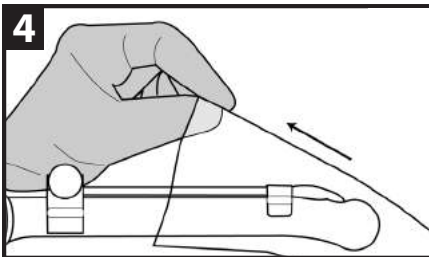
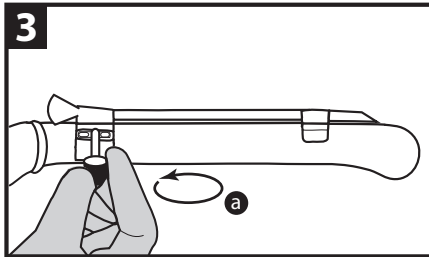
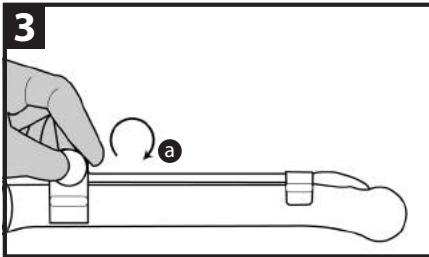
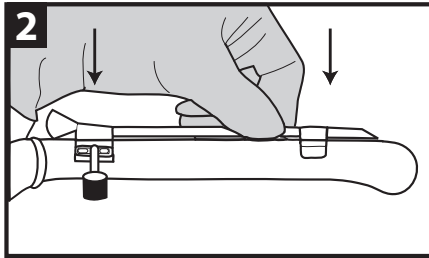
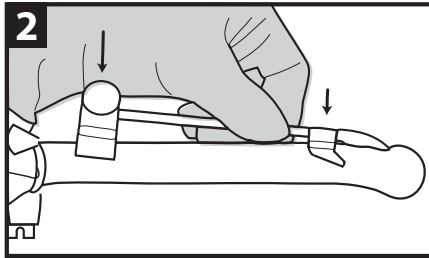
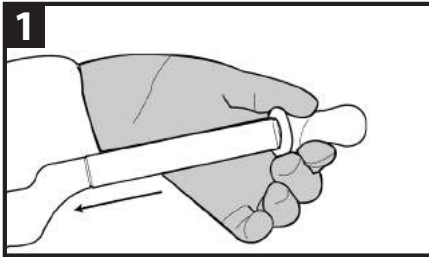
Repeated processing has minimal effect on these devices. End of life is normally determined by wear and damage due to use.

For translations of this instruction, go to www.protekmedical.com.

For more information contact Protek Medical Products, Inc.

Disclaimer:

The instructions provided above have been validated by the medical device manufacturer as being capable of preparing a medical device for re-use. It is the user's responsibility to qualify any deviations from recommended method of processing, and properly evaluate for effectiveness and potential adverse consequences.

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1-041-6004WEB/GER REV 1.2 ISSUED DATE: 02-01-2013 - Copyright © 2012 Protek Medical Products, Inc.
1-041-6004WEB/SPA REV1.12 ISSUED DATE: 02-01-2013 - Copyright © 2012, Protek Medical Products, Inc.
1-041-6004WEB/RUS REV1.12 ISSUED DATE: 02-01-2013 - Copyright © 2012 Protek Medical Products, Inc.