

SONO ULTRASOUND / DISINFECTING WIPES

Manufacturer's Instructions for Use (MIFU)

One Step Cleaner – Disinfectant – Deodorizer - Sanitizer

For Hospitals, Household, Restaurants, Institutional and Industrial Use

- Staphylocidal • Pseudomonacidal • Bactericidal • Salmonellacidal • Fungicidal • Virucidal*

Active Ingredients:

Octyl decyl dimethyl ammonium chloride.....	0.025%
Diocetyl dimethyl ammonium chloride.....	0.010%
Didecyl dimethyl ammonium chloride.....	0.015%
Alkyl (C14, 50%; C12, 40%; C16, 10%) dimethyl benzyl ammonium chloride.....	0.034%
Other Ingredients.....	99.916%
Total.....	100.000%

DIRECTIONS FOR USE:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. Do not use as a diaper wipe or for personal cleansing. This is not a baby wipe! Do not flush in toilet. For plastic and painted surfaces, spot test on an inconspicuous area before use.

This product or product name will not harm most surfaces: including acrylic, sealed fiberglass and vinyl.

Do not use on unpainted wood. Note: Not recommended for unpainted wood, natural marble or brass.

This product is not to be used as a terminal sterilant / high level disinfectant on any surface or instrument that is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. **This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.**

{For Canister}

To Open Package:

Remove cover. Pull up wipe corner from center of roll. Thread through hole in the cover. Replace cover. Open center cap on cover to remove wipe. When not in use, close center cap securely to prevent moisture loss.

{For Soft pack}

To Open Package:

Peel back the label. Remove wipe as needed. To retain moisture, reseal by firmly pressing seal back on the soft pack.

TO CLEAN AND DEODORIZE:

Wipe surface with wipe. Let air dry. Discard used wipe in trash. Do not flush in toilet.

TO CLEAN AND DISINFECT [DISINFECTION / VIRUCIDAL* / MOLD AND MILDEW CONTROL]:

Wipe surface with wipe until surface is visibly wet. Use enough wipes to thoroughly wet surfaces. Use extra wipes to keep surfaces wet for 4 minutes. **For Candida albicans**, treated surfaces must remain wet for 8 minutes. **For Norwalk Virus, Feline Calicivirus, Norovirus and Rotavirus**, treated surfaces must remain wet for 10 minutes. Permit surface to air dry.

If surfaces are extremely dirty, clean first with another wipe before disinfecting.

Discard used wipe in trash. Do not flush in toilet.

Rinse food contact surfaces with clean, potable water after disinfecting. Do not use to disinfect dishes, glassware or utensils.

For mold and mildew, SONO Ultrasound/Disinfecting wipes will effectively inhibit the growth of mold and mildew and the odors caused by them when applied to hard, nonporous surfaces. Follow disinfection instructions. Repeat treatment every seven days, or more often if new growth appears.

SONO ULTRASOUND / DISINFECTING WIPES

Manufacturer's Instructions for Use (MIFU)

***KILLS HIV AND HBV AND HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces / objects with blood or body fluids, and in which the surfaces / objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 [HIV-1] [associated with AIDS] or Hepatitis B Virus [HBV] or Hepatitis C Virus [HCV].

Special Instructions for Cleaning and Decontamination against HIV-1 or HBV or HCV on Surfaces / Objects Soiled With Blood / Body Fluids.

Personal protection: Clean-up must always be done wearing protective gloves, gowns, masks and eye protection.

Cleaning procedure: Blood and other body fluids containing HIV or HBV or HCV must be thoroughly cleaned from surfaces and objects before application of **SONO Ultrasound/Disinfecting wipes**. **SONO Ultrasound/Disinfecting wipes** can be used for this purpose.

Contact time: Leave surface wet for 1 minute [60 seconds] for HIV-1. Use appropriate contact time as listed on label for all other viruses, bacteria and fungi claimed.

Disposal of infectious material: Blood, body fluids, cleaning materials and clothing must be disposed of according to local regulations for infectious waste disposal.

This product is for use on hard nonporous surfaces not conducive to treatment by immersion or excess liquid against Avian Influenza A

SANITIZING [DEODORIZING]:

To sanitize and deodorize hard, non-porous, non-food contact surfaces; wipe surface, using enough wipes to allow treated surface to remain wet for 15 seconds for Staphylococcus aureus [Staph] [ATCC 6538], Klebsiella pneumoniae [ATCC 4352], and Campylobacter jejuni [ATCC 29428]. Let surface dry. For highly soiled surfaces, clean excess dirt first

CAUTION [PRECAUCIÓN] KEEP OUT OF REACH OF CHILDREN

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

Caution. Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco.

FIRST AID

IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then continue rinsing eye.

Call a poison control center or doctor for treatment advice. Have the container or label with you when calling a poison control center or doctor, or going for treatment.

STORAGE AND DISPOSAL

Store in original container in areas inaccessible to children. Keep securely closed. Non-refillable container. Do not reuse or refill this container. Discard empty container in trash or recycle.

Manufactured for:
Advanced Ultrasound Solutions, Inc.
1012 W. Beverly Blvd
Montebello, CA 90640
www.ultrasoundwipes.com
www.sonowipes.com
855-879-4737
EPA Reg. No. 6836-152-89018

